

## CLEANING & DISINFECTING ROHO CUSHIONS

**Cleaning** is the removal of soil, dirt and organic matter from the cushion.

**Disinfecting** is the elimination of pathogenic microorganisms.

***Cleaning must precede disinfection.***

### Cleaning:

- Close the valve(s) and place in a large deep wash basin.
- **Soak cushion in warm soapy water for at least 30 minutes prior to washing and longer if very sticky! DO NOT PULL THE CELLS APART AS THEY ARE VERY FRAGILE!**
- Use a washcloth to gently, but thoroughly scrub all of the air cells and the spaces between.
- Use the edge of the basin to spread the air cells for good access to the base.
- Rinse completely and towel dry.

### Disinfecting:

- Spray Virox over surface of the cushion and allow the Virox to sit on the surface for 10 minutes, then rinse completely and hang to dry. Cushion covers should be washed separately in warm soapy water and rinsed, then spray with Virox solution and let sit for 10 minutes, rinse and hang to dry.

